

Instructions for mouth cell collection (buccal swab sampling)

The dog should have clean mouth. Please, do not serve food or drink at least 1 hour before sampling.

If puppies are still breastfeeding, please take a sample at least 1 hour after last breastfeeding. Puppy should also be during this period in limited contact with its mother and other siblings.

1. Open the package with the sampling swab (brush) placed in a tube.



2. Remove the swab from the tube. Make sure that your fingers do not touch the sampling part of the swab (Fig. 1).

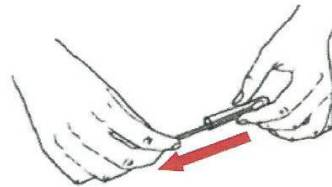


Fig. 1

3. Insert the swab into the mouth of the dog and rub for at least 10 seconds inside cheek of the dog's mouth. When taking the sample, apply constant pressure and rotate the swab regularly. **Try to cover by rubbing maximum area of a dog's mouth.** (Fig. 2).

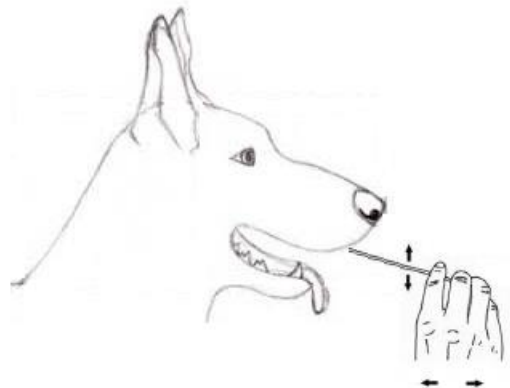


Fig. 2

4. **After swabbing let the buccal swab air dry for at least 1 hour at room temperature.**
5. Put the swab back into the tube. Clearly label the tube with the dog's name. Please, fill out the request order form and send it together with the labeled tube to the laboratory.

Shipping:

Please, use professional courier for shipping (DPD, UPS or others). Note, that if you send a parcel by standard international post, the parcel might be significantly delayed or even lost. If asked by a courier, define the content as "exempt animal specimen - samples".